


LUMILY™

LED UV STERILISATION CASE Instruction Manual

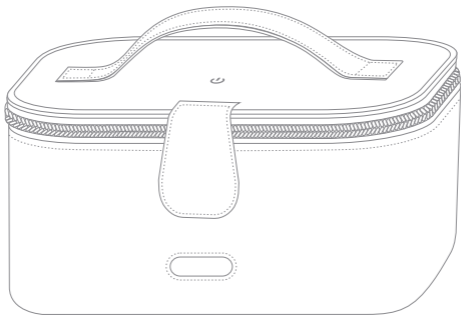

www.lumily.uk

PLEASE READ THIS MANUAL BEFORE USE
& KEEP FOR FUTURE REFERENCE




PRODUCT OVERVIEW

The Lumily Sterilisation Case uses LED ultraviolet disinfection technology to kill 99.9% of bacteria.

TWO MODES

The case features 2 sterilisation modes;

- **Quick Mode:** 3 minute disinfection cycle
- **Boost Mode:** 8 minute disinfection cycle

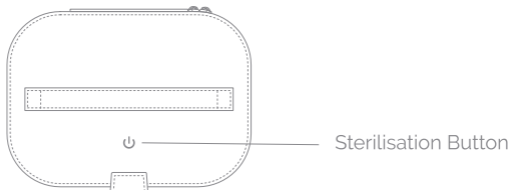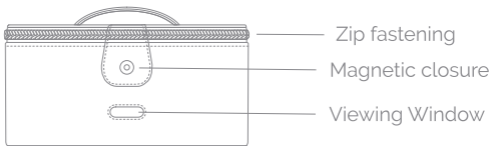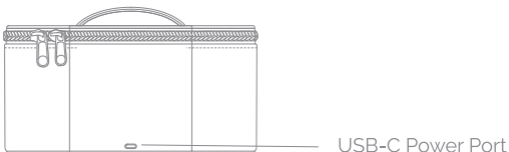

INSTRUCTIONS

TO OPEN

Unzip the case and place your object inside. Ensure that the lid is fastened securely before setting the sterilisation time.

POWER ON

Connect to the power using the USB cable supplied. The power port will glow blue when the power is connected.

TO STERILISE

Select the required sterilisation time;

Quick Mode: to activate the 3 minute quick mode, press the sterilise button once (one beep confirms activation).

Boost Mode: to activate the 8 minute boost mode, press the sterilise button twice (two beeps confirms activation).

During activity the UV light can be seen via the window to the front of the bag. Once the sterilisation time is complete, the light will go out and sterilisation ends. The end of the cycle is represented by three continuous beeps.

STOP OR CHANGE MODE

To stop the current cycle, simply long press the sterilise button. One beep confirms the cycle has stopped. The UV light will go out and sterilisation ends.

SAFETY FEATURE

If the magnetic closure is released during sterilisation the cycle will be stopped temporarily. If the magnet reconnects the cycle will continue. To completely stop the cycle long press the sterilise button.

SPECIFICATION

MODEL	UV.VC.003/004
OUTPUT	3W (max)
INPUT POWER	5V/1A USB-C
UV LED LIFESPAN	10,000 hours
UV WAVELENGTH	270-280nm
SIZE	257 x 208 x 114mm
MATERIAL	PU
CONTENTS	Sterilisation Case USB Cable Instruction Manual
GUARANTEE	See website for details



WARNING

Do not expose to eyes and skin



RoHS
COMPLIANT



SAFETY PRECAUTIONS

- Ensure that you use the power supply as recommended.
- Never disassemble the UV LED lights, these are non-replaceable.
- Never use under water or beyond the specified range of use.
- Always use on a flat, secure surface.
- Do not move during use, this may cause damage to the case or the items inside.
- Always ensure the lid is firmly closed during sterilisation to avoid UV leakage. UV radiation causes damage to eyes and skin. Avoid exposure of eyes and skin.
- Never use in a strong magnetic environment.
- To secure the lid properly, ensure that no foreign objects are causing an obstruction. Ensure the magnet closure is secured properly.
- If product fails to work normally, disconnect from the power source immediately and contact the manufacturer.
- To avoid potential hazards, never disassemble or repair this product.
- Do not leave product unattended during use.
- Store safely, out of reach of children.

www.lumily.uk

LUMILY LTD WF10 5PY

CLEANING & MAINTENANCE

UNPLUG POWER CABLE

Make sure to disconnect from the power source when cleaning.

WIPE CLEAN

Wipe throughout with a soft dry cloth before and after use. Do not use abrasive materials to clean this product as this will cause damage to the sterilisation components.

REMOVE ITEMS

Do not store items, remove when not in use.

DO NOT IMMERSE IN WATER

Never clean this product with water or liquids.

TURN OFF POWER

Ensure to turn off the power when the product is not in use and unplug the USB cable for safety.

NOTE:

Not suitable for liquid and liquid items

Not to be used for food storage